


### 1. Overview

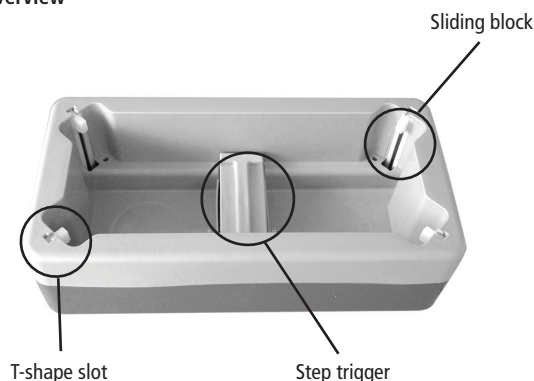

### 2. How to load shoe cover

1.



Inserting shoe covers. Make sure that the length and width of the overshoe matches the dispenser's length and width dimensions. Spread the first roll of five or ten new overshoes and insert them directly into the T-shape slot located in each corner of the machine.

2.



Start with one corner and insert the overshoes with the T-shape into the slot. Then put the second T-shape in the corner diagonally located. Repeat this for the last two corners.

3.



After inserting the first roll, remove all ties.

4.



Before inserting the second roll, the first roll needs to be pushed into the slot. Then please repeat step 1-3 for inserting the overshoes.

5.



Make sure that the overshoes are sitting flat and even inside the overshoe machine.

6.



Pull out the first overshoe at the corner of the T-shape slot gently, approximate 1mm, until it locks into place.

### 3. How to use

- Place your foot with a flat and firm step into the machine.
- When you push down with your foot the overshoe machine will automatically place a clean cover over your shoe.
- Lift your foot out and repeat with the other foot.



step 1



step 2



step 3

### Notice

It is recommended to load 30-40 shoe covers at a time.